

Huve-Check® Crypto strips

Test Strips for detection of
Cryptosporidium parvum in faeces


HOW TO PROCEED



- Take the faeces directly from the rectum. If the samples are liquid, take a spoonful.




- If the samples are solid, remove the excess amount with a spatula or clean object.

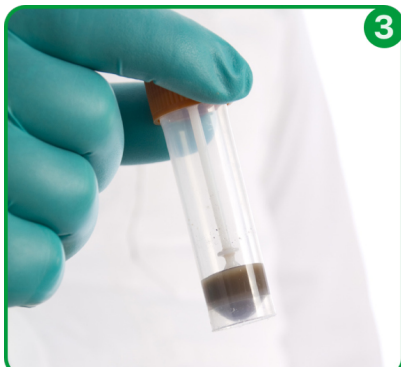



- Dilute the sample with the liquid contained in the bottle. Homogenize well, taking care to prevent foam formation.



- Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.

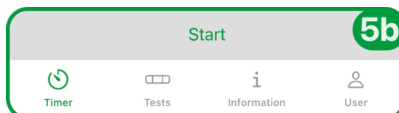

- Plunge a device into the liquid and start the timer (showing this device to your phone camera).

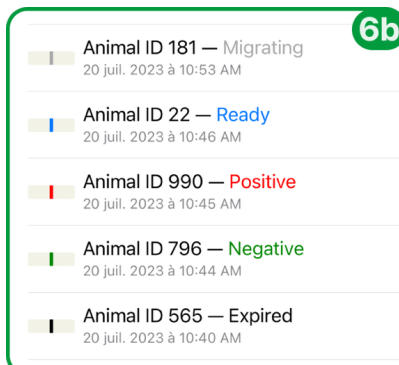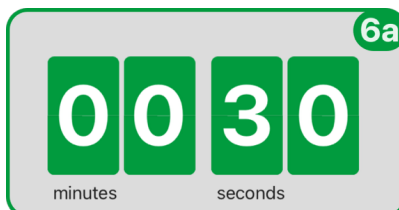



- Wait for about 10 minutes, until the test is ready, then interpret **immediately**.

Interpretation with app



- Scan the result (touch the added test entry) in good lighting conditions.

OR Interpretation with eyes

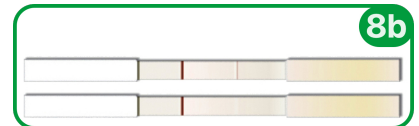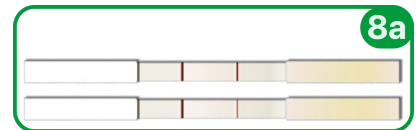

- If the image quality is still not good enough after 3 attempts (⚠️⚠️⚠️), analyse according to the examples above.

- 8a : positive result;
- 8b : negative result.

Precautions for use

- Bring all components between 18°C (64°F) and 24°C (75°F).
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.
- Store kits in a dry place between 4°C (39°F) and 30°C (86°F).

