


Huve-Check® Crypto

(1 X 10 strips)

For the detection of Cryptosporidium

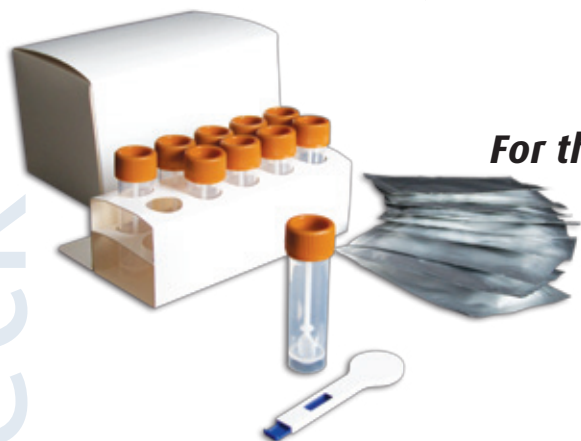

How to proceed



1. Take a sample of stool using the enclosed measuring spoon. If the samples are liquid, collect one level spoonful.



2. Dilute with the liquid contained in the bottle, if the samples are solid remove the excess amount with a spatula or clean object.




3. Homogenize well, taking care to prevent foam formation.



4. Plunge a strip into the diluted sample. The red portion of the strip must not be immersed into the liquid



5. Wait 10 minutes and interpret the results. To safeguard the reagents' stability, take care to close the tube containing the strips immediately.

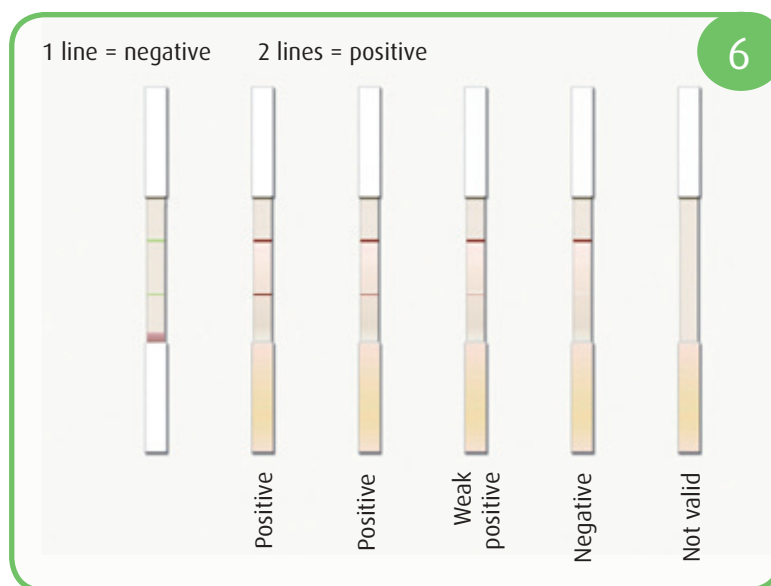

6. Results

Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.

