



Huve-Check® Calf scours 5

Strips for detection in calf faeces:

- ▶ Rotavirus
- ▶ Coronavirus
- ▶ *E. coli* Attachment Factor F5(K99)
- ▶ *E. coli* CS31A
- ▶ Cryptosporidium



How to proceed

1

Take the faeces directly from the rectum of the calf. If the samples are liquid, take a spoonful.



2

If faeces are solid, remove the excess amount using a spatula.




3

Dilute its content in the liquid of the small tube called sample tube.



4

Insert the sample tube into the strip tube.

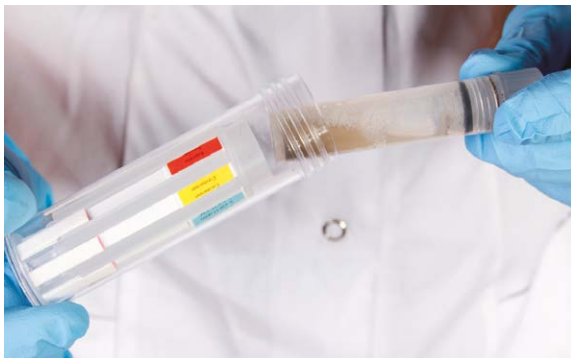


5

Screw the top of the strip tube. You must hear two separate clicks. Leave the device vertically on a flat surface and wait for 10 minutes.



6

Results

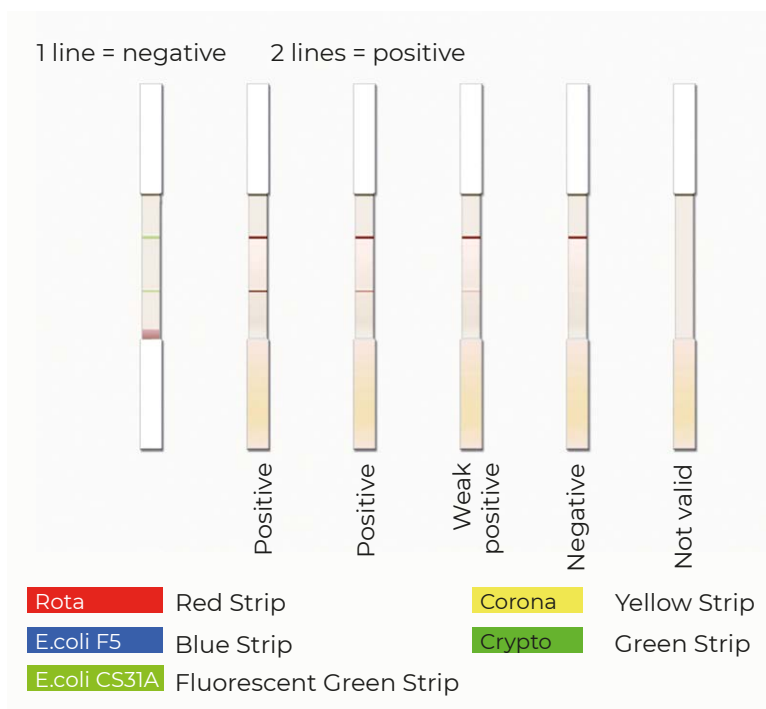

Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.

Huvepharma EOOD. 3a Nikolay Haytov Str, 1113 Sofia Bulgaria.
Tel +359 2 862 5331. sales@huvepharma.com

Huvepharma N.V. Uitbreidingstraat 80, 2600 Antwerp, Belgium.
Tel +32 3 288 1849. customerservice@huvepharma.com